

Service Bulletin

Tonometers & Mounts

For Xcel Slit Lamps

#14-12590-01
October 15, 2014

The chart below is to clarify the different available mounts that are needed to accommodate the CT100 and CT210 tonometers for the various Xcel slit lamps.

The Xcel 400/455/700 lamps all use the CT100 tonometer. All of these lamps use the same 12593 mount, except a select few late model Xcel 400/700's that have a SN of 22805-0613 and higher. Those lamps use the unique 12599 mount.

The Xcel 200/250/255 lamps all use the CT210 tonometers. These all use the 12576 mount, except for the Xcel 255. The Xcel 255 uses the 12597 mount. Both of these mounts appear identical, however, the base of the mounts have different thicknesses. This difference makes it imperative to use the proper mount with the corresponding lamp. The tonometer will not align properly if using an incorrect mount.

Slit Lamp	Tonometer	Mount P/N
Xcel 200/250	CT210	12576
Xcel 255	CT210	12597
Xcel 400/700	CT100	12593
Xcel 400/700 (SN's 22805-0613 and up)	CT100	12599
Xcel 455	CT100	12593



CT100 (P/N 12591)



12593



12599



CT210 (P/N 12598)



12576



12597



Reichert, Inc. 3362 Walden Avenue, Depew, NY 14043

Toll Free: 888-849-8955 • Tel: (716) 686-4500

Fax: (716) 686-4545 • www.reichert.com • ISO 9001/13485 Certified

12590-104 RevA

